

THE MINI MAT™

FOR STEP STOOL APPLICATION

Cat.# USA-A10 (10 mats/case)

13in x 17in

(8 8 8) 6 3 3 - 5 8 4 3

DIRECTIONS FOR USE

1) Remove *THE MINI MAT™* from the packaging and determine the proper position for the Mat on the step stool (standing stool, foot stool, riser, etc.). The surface where the Mat will lay **MUST** be completely dry and clean prior to placement. Be sure all wording on the Mat is facing toward the user so it can be read by the user while facing the surgical table.

IMPORTANT NOTE: Make sure the Mat will not overhang on the sides of the step stool in order to prevent any chance of tripping or slipping. Do not use the Mat if overhang is noticed. Do not cut, tear or alter the Mat to fit the step stool because this will damage the use and function of the Mat. If the Mat does not fit the step stool, do not use it.

2) Remove the colored adhesive strip tab on the underside of the Mat to expose the adhesive strip and place the Mat on the step stool with the adhesive strip side facing down. Apply firm pressure to ensure the adhesive strip is secured to the step stool.

IMPORTANT NOTE: Adhesive strip must always be used when using the Mat to prevent slippage.

3) The Mat is now ready for use. **IMPORTANT NOTE:** If the Mat is picked up and repositioned once it has been adhered to the step stool, the adhesive strip will not function properly and the Mat should be discarded and a new Mat should be used.

4) At the conclusion of the surgical procedure, lift the Mat off of the step stool and discard the Mat with all other contaminated items, pursuant to hospital regulations and guidelines. **IMPORTANT NOTE:** Never attempt to reuse the Mat.

NOTE: The MINI MAT is not designed to absorb fluids, but rather channel fluids away from the surgeons/assistants standing on the Mat(s).

CAUTION: Do not use the Mat on uneven, textured or non-skid surfaces as slippage or tripping may occur causing an injury.

THE MINI MAT™

is a One-Time Use, Disposable Product!

It **cannot** be cleaned and should never be re-used to ensure avoidance of cross-contamination throughout the operating room.

FLAGSHIP SURGICAL, LLC
16 Mt. Bethel Road, Suite 313
Warren, NJ 07059
www.flagshipsurgical.com



Manufactured by an
ISO 9001:2000 Certified Company



FLAGSHIP SURGICAL

THE MINI MAT™

FOR STEP STOOL APPLICATION

Cat.# USA-A10 (10 mats/case)

13in x 17in

(8 8 8) 6 3 3 - 5 8 4 3

Additional Terms

These terms and conditions apply to your ("Customer") purchase and use of either "The Surgical Mat™" or The Mini Mat™ (collectively, the "Mat") provided by Flagship Surgical, LLC (the "Company").

WARRANTY. For a period of forty-five (45) days after delivery of the Mats to Customer (the "Warranty Period"), the Company warrants that the Mats will be free from any material defects when used in accordance with the provided instructions. Customer shall notify the Company in writing of any warranty claim and if the warranty claim is properly made during the Warranty Period, the Company's sole obligation, and Customer's exclusive remedy shall be for the Company to replace the defective Mats or, in the Company's sole discretion, provide Customer with a full refund of the purchase price paid for the defective Mats.

DISCLAIMER OF WARRANTIES. EXCEPT AS OTHERWISE SET FORTH HEREIN THE MATS ARE PROVIDED "AS IS" WITHOUT ANY WARRANTIES OF ANY KIND AND THE COMPANY HEREBY DISCLAIMS ALL OTHER WARRANTIES, WHETHER EXPRESS OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. The limited warranty set forth above gives Customer specific legal rights. Customer may have additional rights that vary from jurisdiction to jurisdiction. THE COMPANY SHALL NOT BE LIABLE FOR ANY USE OF OR CIRCUMSTANCES RELATING TO CUSTOMER'S IMPROPER USE OF THE MATS, INCLUDING, BUT NOT LIMITED TO, CUSTOMER'S OWN NEGLIGENCE.

LIMITATIONS OF LIABILITY. IN NO EVENT WILL THE COMPANY'S CUMULATIVE LIABILITY EXTEND BEYOND CUSTOMER'S ACTUAL, DIRECT, PROVABLE DAMAGES IN AN AMOUNT NOT TO EXCEED THE AMOUNT PAID BY CUSTOMER FOR THE MATS. IN NO EVENT WILL THE COMPANY BE LIABLE FOR ANY PUNITIVE, AGGRAVATED, EXEMPLARY, SPECIAL, CONSEQUENTIAL, INCIDENTAL OR INDIRECT DAMAGES OR ANY LOSS OF PROFITS, LOSS OF BUSINESS OR GOODWILL HOWEVER CAUSED, WHETHER FOR BREACH OF WARRANTY, BREACH OF CONTRACT, TORT, NEGLIGENCE, OR OTHERWISE, EVEN IF THE COMPANY HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

GOVERNING LAW. The terms hereof shall be governed by and construed in accordance with the laws of the State of New Jersey, without regards to its conflicts of law principles.

CONFLICTS. In the event Customer issues a purchase order in connection with the purchase of the Mats, the terms and conditions contained herein shall control if such purchase order contains terms and conditions that are inconsistent with or conflict with the terms contained herein.

FLAGSHIP SURGICAL

