

THE SURGICAL MAT[™]

Cat.# USA-010

(10 mats/case)

20in x 39in

(888)633-5843

PATENTED

DIRECTIONS FOR USE

- 1) Remove *THE SURGICAL MAT*[™] from the packaging and determine the proper position on the operating room floor. The floor MUST be completely dry and clean prior to the placement of the Mat on the floor. Be sure that all wording on the Mat is facing toward the user so it can be read by the user while facing the surgical table.
- 2) Once the correct location on the floor has been identified, remove both colored adhesive strip pull tabs on the underside of the Mat and place the Mat on the dry, clean floor. Apply firm pressure to ensure both adhesive strips are secure to the floor.

Important Note: Both adhesive strips must always be used when using the Mat to prevent slippage.

- 3) The Mat is now ready for use. **Important Note:** If the Mat is moved or repositioned during the surgical procedure, the adhesive strips will not function properly and the Mat should be discarded and a new Mat should be used.
- 4) At the conclusion of the surgical procedure, lift the Mat from one end and gently pull it off of the floor. Roll up the Mat and discard it with all other contaminated items, pursuant to hospital regulations and guidelines. **Never attempt to reuse the Mat.**

Note: For fluid-intense procedures, it is recommended to place suction devices on both sides of the Mat to reduce fluid build-up on the floor. The Mat is not designed to absorb fluids, but rather to channel fluids away from the surgeons/assistants standing on the Mat(s).

CAUTION: Do not use the Mat on uneven, textured or non-skid surfaces as slippage may occur causing an injury.

THE SURGICAL MAT[™]

is a One-Time Use, Disposable Product!

It **cannot** be cleaned and should never be re-used to ensure avoidance of cross-contamination throughout the operating room.

FLAGSHIP SURGICAL, LLC
16 Mt. Bethel Road, Suite 313
Warren, NJ 07059

www.flagshipsurgical.com



Manufactured by an
ISO 9001:2000 Certified Company



FLAGSHIP SURGICAL



LATEX FREE

NON-STERILE



90°F / 32°C
50°F / 10°C



SINGLE USE ONLY

THE SURGICAL MAT™

Cat.# USA-010

(10 mats/case)

20in x 39in

(888)633-5843

PATENTED

Additional Terms

These terms and conditions apply to your (“Customer”) purchase and use of either The Surgical Mat™ or The Mini Mat™ (collectively, the “Mat”) provided by Flagship Surgical, LLC (the “Company”).

WARRANTY. For a period of forty-five (45) days after delivery of the Mats to Customer (the "Warranty Period"), the Company warrants that the Mats will be free from any material defects when used in accordance with the provided instructions. Customer shall notify the Company in writing of any warranty claim and if the warranty claim is properly made during the Warranty Period, the Company's sole obligation, and Customer's exclusive remedy shall be for the Company to replace the defective Mats or, in the Company's sole discretion, provide Customer with a full refund of the purchase price paid for the defective Mats.

DISCLAIMER OF WARRANTIES. EXCEPT AS OTHERWISE SET FORTH HEREIN THE MATS ARE PROVIDED “AS IS” WITHOUT ANY WARRANTIES OF ANY KIND AND THE COMPANY HEREBY DISCLAIMS ALL OTHER WARRANTIES, WHETHER EXPRESS OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. The limited warranty set forth above gives Customer specific legal rights. Customer may have additional rights that vary from jurisdiction to jurisdiction. THE COMPANY SHALL NOT BE LIABLE FOR ANY USE OF OR CIRCUMSTANCES RELATING TO CUSTOMER'S IMPROPER USE OF THE MATS, INCLUDING, BUT NOT LIMITED TO, CUSTOMER'S OWN NEGLIGENCE.

LIMITATIONS OF LIABILITY. IN NO EVENT WILL THE COMPANY'S CUMULATIVE LIABILITY EXTEND BEYOND CUSTOMER'S ACTUAL, DIRECT, PROVABLE DAMAGES IN AN AMOUNT NOT TO EXCEED THE AMOUNT PAID BY CUSTOMER FOR THE MATS. IN NO EVENT WILL THE COMPANY BE LIABLE FOR ANY PUNITIVE, AGGRAVATED, EXEMPLARY, SPECIAL, CONSEQUENTIAL, INCIDENTAL OR INDIRECT DAMAGES OR ANY LOSS OF PROFITS, LOSS OF BUSINESS OR GOODWILL HOWEVER CAUSED, WHETHER FOR BREACH OF WARRANTY, BREACH OF CONTRACT, TORT, NEGLIGENCE, OR OTHERWISE, EVEN IF THE COMPANY HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGE.

GOVERNING LAW. The terms hereof shall be governed by and construed in accordance with the laws of the State of New Jersey, without regards to its conflicts of law principles.

CONFLICTS. In the event Customer issues a purchase order in connection with the purchase of the Mats, the terms and conditions contained herein shall control if such purchase order contains terms and conditions that are inconsistent with or conflict with the terms contained herein.

FLAGSHIP SURGICAL



NON-STERILE



SINGLE USE ONLY